
CARE & MAINTENANCE OF VENTURI NPWT SYSTEMS

Power Unit

Always disconnect the power unit from the electricity supply (if applicable) before cleaning. The power unit can be wiped down with a damp cloth or alcohol wipe. Do not use solvents. CAUTION: Do not immerse in water.

The unit contains no user serviceable parts, and should only be serviced by a competent electrical engineer, or returned to Talley Medical, or your local authorised dealer.



Canisters

Canisters are disposable and intended for single use only. Dispose of used canisters according to local clinical waste policy.

Carry Bag

For day to day cleaning purposes it is suggested that bags are cleaned using hot water and soap / neutral detergent, or chlorine releasing agents at a concentration of less than 1000 ppm. The carry bag can also be machine washed at temperatures not exceeding 71°C. Drip dry only, do not tumble dry. Do not use bleach, chlorine releasing agents in concentrations over 1000 ppm, solvents or alcohol-based cleansers, e.g. Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline as these may damage the carry bag materials. Alcohol wipes may be used sparingly. Do not Autoclave.



Bedside Holder

The bedside holder can be wiped down with a damp cloth or alcohol wipe. Do not use solvents.

